

DISPOSABLE BIPOLAR PENCILS

Instructions for Use – Single Use Bipolar Pencil

This product is disposable and is supplied sterile. Prior to the expiration date, contents are sterile if the package has not been opened or is not damaged.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

INDICATIONS: The disposable pencils are electro-surgical devices designed to be used in soft tissue surgical procedures.

WARNING: CONNECT BIPOLAR ACCESSORIES TO THE BIPOLAR RECEPTACLE AND MONOPOLAR ACCESSORIES INTO THE MONOPOLAR RECEPTACLE. IMPROPER CONNECTION OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.

Attach the sterile pencil to the sterile cable ensuring that the pencil pins are fully seated in the cable receptacles. The cables to the surgical electrodes should be positioned in such a way that contact with the PATIENT or other leads is avoided. Temporary unused ACTIVE ELECTRODES should be stored isolated from the patient.

Connect the sterile cable to the generator only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

CAUTION: BECAUSE OF THE VARIABILITY OF OUTPUT VOLTAGES AND MODES FROM GENERATOR TO GENERATOR, DO NOT USE THE PENCIL WITH GENERATORS HAVING BIPOLAR OUTPUT VOLTAGES THAT EXCEED 550 VPK. REFER TO THE APPROPRIATE ELECTROSURGICAL GENERATOR MANUAL FOR INDICATIONS AND INSTRUCTIONS ON BIPOLAR OUTPUT CHARACTERISTICS TO ENSURE THAT ALL SAFETY PRECAUTIONS AND INSTRUCTIONS ARE FOLLOWED.

At the lowest power setting test the pencil by pressing the generator's activating switch. If the generator fails to activate, check the pencil connection with the cable.



SEE INSTRUCTIONS
FOR USE

LOT

BATCH CODE

MANUFACTURED FOR: Ambler Surgical
404 Gordon Drive
Exton, PA 19431
Phone: 1-888-407-0006
www.AmblerSurgical.com



USE BY:
YEAR / MONTH

REF

CATALOG NUMBER



SINGLE PATIENT
USE

STERILE R

STERILE-STERILIZED
BY IRRADIATION

Sterility guaranteed unless package opened
or damaged. Do not resterilize.

Rx Only

CAUTION: Federal (USA) law restricts
this device to sale by or on the order of
a physician.