ellman®

Reusable High Frequency
Cables, Handpieces,
Electrodes and Accessories

Rx
ONLY

Any minor to severe serious injury/incident which has occurred in relation to the device usage/operation should be reported to manufacturer and competent authority of EU member state where operational service is being performed or where the user and/or patient is established.

Safety and Maintenance of Reusable High Frequency Cables / Handpieces / Electrodes / Accessories

WARNING: Electrical contacts must be completely dry prior to connection with other electrical equipment and instruments. Use of damaged equipment or wet contacts may cause electrical burns to the patient, staff, or doctor.

Note: Do not use dry heat as product damage may occur.

Refer to generator instruction manual for more detailed procedural instructions.

HANDLING

1. Inspect prior to each use for any sign of damage.
2. Immediately replace any product which has visible damage or is not functioning properly.
3. Reusable items have a limited life span which may be prolonged by careful handling during procedures and sterilization.
4. Avoid sharply bending, kinking or puncturing insulation. Broken wires may result in intermittent or permanent malfunction.
5. Disconnect cables from generator by pulling on plug and not cable.
6. Electrodes should only be handled by the insulated shafts, the surgical grade wires have thin diameters and are very fragile.
7. When using bendable electrodes, bend only at insulated shaft.
8. Tissue should be moist for cutting and coagulation.
9. Pressure is not required for cutting and coagulation.
10. When used for cutting and coagulation, activate prior to making tissue contact.

Storage

1. Store cables loosely coiled (4-5" diameter).
2. Do not kink cables.
3. Do not set heavy objects on devices.

Cleaning Electrodes During Procedure

1. Set the Radio Surgical unit at cut or cut/coag waveform.
2. Place four gauze pads together and fold in half.
3. Saturate the gauze with water.
4. With the electrode in handpiece, place it between the folds of the gauze.
5. Activate the unit for 1-2 seconds while moving the electrode back and forth within the gauze with little or no finger pressure. Repeat procedure (you should hear "sizzling" sound).

Cleaning Electrodes After Procedure

1. Always clean and disinfect devices.
2. Prior to sterilization, clean cables with a damp cloth moistened in a mild detergent water solution.
3. Avoid immersion in any solution.

Note: No scalpel, scissors or similar pointed objects should be used for cleaning the electrodes, since they may damage the electrode surface. Such damage increases adhesion of tissue to the surfaces of the electrodes during use.

STERILIZATION

Note: When sterilizing Handpieces, be sure to open or loosen the top (Chuck) portion of the Handpiece.

Steam Sterilization:

1. Clean the product thoroughly.
2. Carefully place the device in a sterilization container, avoiding contact with metal instruments, metal trays, or metal chamber walls.
3. Steam Gravity cycle 134° for 18 minutes. Steam Pre-Vacuum cycle 134° C for 4 minutes.
4. When steam sterilization cycle is complete, remove container and allow to cool to room temperature.
5. Thorough drying of electrical contacts is essential.
6. Reusable high frequency cables, handpieces, electrodes and accessories are designed to last a minimum of 20 autoclave cycles. Inspect the product prior to each use; discard if damaged.

DISINFECTION

Chemical Disinfection: Only EPA hard surface disinfection solutions capable of high level disinfection, e.g. 2% glutaraldehyde, should be used and manufacturer's recommendations must be followed. Solutions with a strong acidic or alkaline pH are not recommended.

1. Clean thoroughly.
2. Follow solution manufacturers' instructions paying close attention to proper dilution and recommended minimum exposure times to attain disinfection.
DO NOT EXCEED RECOMMENDED MAXIMUM EXPOSURE TIME OR 45 MINUTES, WHICH EVER IS LESS.
3. After removal from the disinfection solution, rinse thoroughly with sterile water, particularly the electrical contact areas of the connector.
4. Thorough drying of the electrical contacts of the cables / handpieces is essential.

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Cynosure, LLC