REUSABLE OPHTHALMIC BURR TIPS EN

REF 0011, 0012 (For use with REF 0010 Burr Handle)

WARNING: (THIS ONLY APPLIES WHEN USING BURR HANDLE)

Do not use in the presence of flammable gases/materials or oxygen-enriched environments. Fire could result.

CAUTION:

This product is sold NON-STERILE. Sterilize prior to use.

CLEANING INSTRUCTIONS:

- Remove all gross matter (blood, tissue) by wiping the burr using a gauze pad or soft cloth dampened with warm tap water until all visible debris is removed.
- Soak the burr in an enzymatic detergent mixture diluted according to the manufacturer’s instructions. The burrs may also be cleaned using a soft bristled cleaning brush to remove any visible debris.
- Remove cleaning agents from burr by wiping with a cloth dampened in tap water.
- Dry burr thoroughly.

STERILIZATION INSTRUCTIONS:

NUMBER OF USES WHEN STERILIZED BY STEAM AUTOCLAVE: 25

- Steam autoclave at 250° F. to 254° F. (121° C. to 123° C.) for 35 minutes using an autoclavable pouch or wrapped cloth method. Consult the AAMI recommended practice for steam sterilization using the wrapped cloth method. Allow the electrode to cool at least 30 minutes.
 - Flash autoclaving is not recommended.
- Disposal
- Biohazard – Dispose of electrode according to facility procedures

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Bovie Medical Corporation 5115 Ulmerton Road Clearwater, Florida 33760 USA