VFD	Disposable Direct Image Flat Vitrectomy Lens
VMD	Disposable Direct Image 1.5x Magnifying Vitrectomy Lens
VWFD	Disposable Direct Image Wide Field Vitrectomy Lens
V30PD	Disposable Direct Image 30° Prism Vitrectomy Lens
VBCD	Disposable Direct Image Biconcave Vitrectomy Lens
VFLATSSVD	Direct Image Disposable

The vitrectomy lens is a single-use device that helps improve accuracy and clarity of images of the posterior and peripheral retina and of the vitreous body during surgical intervention.

The lens is installed in the cornea and helps increase magnification.

This product is intended for all private or public centers where eye surgery is practiced.

It is intended for use by authorized and trained staff.

PRECAUTIONS

 **RINSE THE LENS WITH STERILE WATER BEFORE USE**

- Do not re-sterilize. For single use only. Use on one patient only.
- Do not use if one of the sachets is damaged.
- Fragile: avoid shocks and handle with care.
- Do not use beyond the sterilization expiry date indicated after the symbol.
- It is preferable to use a viscous interface solution between the cornea and the device.
- This device is to be used by or on the order of a physician.

 Do not re-sterilize. For single use only. Use on one patient only.

 Do not use if package is damaged.

 Do not use if package is damaged.

 Do not use beyond the sterilization expiry date indicated after the symbol.

 Sterile using Ethylene Oxide.

CONTRAINDICATIONS

- Do not re-sterilize the product.
- Do not repair a damaged product.
- Do not use this product if the packaging is damaged.
- Do not immerse the device in a decontaminating product before use.

RISKS

- If the lens is reused, there is a risk of biocontamination for the patient

LIMIT OF LIABILITY

Volk shall not be responsible for loss of income or profit resulting from downtime costs or loss of equipment use.

If you have questions regarding Volk's Warranty, please contact Volk Optical.

<p>Mfd in France for: VOLK Optical Inc. 7893 Enterprise Drive, Mentor, OH 44060 Tel: 440-942-6161 / 800-345-8655 Fax: 440-942-2257 http://www.volk.com • e mail: volk@volk.com</p>	<p>EU REPRESENTATIVE </p> <p>The Volk authorized representative based in the European Union (EU) is: Keeler Limited Clewer Hill Road Windsor Berkshire SL4 4AA U.K. Tel: +44(0) 1753 8 57177</p> <p> 0086</p>
--	--

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Copyright © 2010 Volk Optical Inc.